# VARIGRIP<sup>TM</sup> UNO

Variable Tension Exerciser for Thumb & Individual Fingers

## INSTRUCTION MANUAL

# THANK YOU FOR PURCHASING VARIGRIP UNO!

The following exercises are not tailored to any specific sport, but instead are designed to maintain and improve dexterity, coordination, flexibility, strength, endurance, circulation and general health of your fingers, thumbs and/or hands. VariGrip Uno comes with 1 extra exercise loop and 2 springs (the RED spring is located inside the unit).

**RED:** For extra light, light and medium resistance levels. 1.5-5 lbs./.68-2.27 kg.

**GREEN:** For medium, heavy and extra heavy resistance levels. 5-9 lbs./2.27-4.08 kg.

The area above the tension dial includes visual indicators for each of the resistance levels.

## INSTRUCTIONS:

To increase resistance level, turn resistance knob counter-clockwise. It is important to begin all exercises slowly and to use low resistance with few repetitions. Hold each position for 3-5 seconds, then relax. Repeat 5-10 times. Over time, increase to 3 sets of 10 (30 repetitions total). When graduating to a higher resistance level, begin with 5-10 repetitions and build slowly.

**\*\*NOTE:** VariGrip Uno is an exercise product. Please consult with a health care professional before starting any type of exercise program. Stop use immediately and consult with a health care professional if you feel any pain or discomfort while using the product. Do not use this product in any manner that may cause them to snap towards the head and cause injury to eyes.

### 1. Thumb TIP Flexion

Hold the UNO in your hand (grasp) with the side of your index finger resting under the flexion stabilizer (flared portion). Place your thumb tip on the flexion plunger and bend ONLY the tip of the thumb to compress the plunger, hold 3-5 seconds, then relax. Repeat 10 times slowly, or as instructed by a healthcare professional.

### 2. Thumb MP Flexion

Hold the UNO, under the stabilizer in the web space between your ring and small fingers. Place the bottom of the first thumb (IP) joint on the flexion plunger and bend (flex) ONLY the 2nd joint (MP) of the thumb. Keep the IP joint straight. Hold 3-5 seconds, then relax. Repeat 10 times slowly, or as instructed by a healthcare professional.

### 3. Tripod Pinch (II-III Finger Tips)

Hold the UNO with the loop end upward. Place the UNO between the index and long (middle) finger with the tips resting on the flexion stabilizer. Place the thumb tip on the flexion plunger. While keeping all the joints slightly bent (flexed) move the finger tips toward the thumb tip and pinch. Hold for 3-5 seconds, then relax. Repeat 10 times slowly, or as instructed by a healthcare professional. Try to maintain a C position.

### 4. Tip Pinch (IV-V Finger Tips)

This is similar to the Tripod Pinch above but uses the ring and small finger (pinky) tips. Hold the UNO with the loop end upward. Place the UNO between the ring and small finger with the tips resting on the flexion stabilizer. Place the thumb tip on the flexion plunger. While keeping all the joints slightly bent (flexed), move the finger tips toward the thumb tip and pinch. Hold for 3-5 seconds, then relax. Repeat 10 times slowly, or as instructed by a healthcare professional. Try to maintain a C position.

### 5. Thumb Opposition

Hold the UNO in the web space between the ring and small fingers under the flexion stabilizer – Place the thumb tip on the flexion plunger and bend the two thumb joints while keeping the thumb as far away from the palm as possible for opposition. Hold 3-5 seconds, then relax. Repeat 10 times slowly, or as instructed by a healthcare professional.

### 6. Thumb - Composite - Full Flexion Toward Palm

Hold the UNO in your hand (grasp with ring and small fingers) with the side of the ring finger resting under the flexion stabilizer (flared portion). Place the thumb tip on the flexion plunger and bend BOTH thumb joints to compress the plunger, hold 3-5 seconds, then relax. Repeat 10 times slowly, or as instructed by a healthcare professional.

### 7. Finger Hook Flexion

Hold UNO loosely in your palm so that the index finger tip can rest on top of the flexion plunger. The other fingers will be under the flexion stabilizer (flared portion). You can also use the other hand to help stabilize the UNO base. Bend (flex) the index finger tip to compress the plunger. Both joints will bend. Hold for 3-5 seconds, then relax. Repeat 10 times slowly, or as instructed by a healthcare professional.

( CONTINUED )

### **8. EDC – Full Finger Extension (USE EXERCISE LOOP)**

Put the specific finger you want to exercise all the way through the exercise loop. Hold UNO loosely with your other fingers with the exercise loop at the top. Start with the two smaller joints straight and the base of the finger bent (flexed) as much as you can. You can also use your other hand to help stabilize the UNO. Without moving the entire arm or hand, just raise the finger as if to point direction hold the finger straight for 3-5 seconds, then relax back into starting position. Repeat 10 times slowly, or as instructed by a healthcare professional.

### **9. Intrinsic – PIP/DIP Finger Extension (USE EXERCISE LOOP)**

Put the fingertip you want to exercise into the exercise loop. Hold UNO loosely with your other hand with the exercise ring at the top. Start with the two small joints bent (flexed) and the base of the finger straight (extended). Without moving the entire arm or hand, straighten the two smaller joints as if to point direction. Hold the finger straight for 3-5 seconds, then relax finger back to starting position. Repeat 10 times slowly, or as instructed by a healthcare professional.

### **10. Tip to Tip Pinch and Pull (SWITCH FINGERS, USE EXERCISE LOOP)**

Hold UNO in the hand you are NOT exercising with the exercise loop to the side. Put the finger you want to exercise into the loop. Pull UNO so there is a little tension on the loop. While holding the finger straight, move the finger away from the UNO and hold 3-5 seconds. Repeat 10 times slowly, or as instructed by a healthcare professional. SWITCH direction of the UNO and repeat. Do this with each finger.

### **11. Tripod Pinch – on Finger (USE EXERCISE LOOP)**

Hold UNO in the hand you are NOT exercising with the exercise loop facing upward. Place the tips of 3 fingers you are exercising on the edges of the loop. While keeping the space between the three fingers rounded (all joints slightly bent) pinch the ring to flatten it, while pulling the loop away from the other hand. Hold for 3-5 seconds, then relax. Repeat 10 times slowly, or as instructed by a healthcare professional.

### **12. Finger Extension Flicks (USE EXERCISE LOOP)**

Hold UNO in the hand you are NOT exercising with the exercise loop facing upward. Make a loose fist with the hand you are exercising and hold the UNO near the fist. Straighten (extend) a finger and flick the ring with it, repeating 10 times slowly. Relax, then repeat 10 times more, or as instructed by a healthcare professional.

### **13. Finger Flexion Flicks (USE EXERCISE LOOP)**

Hold UNO in the hand you are NOT exercising with the exercise loop facing upward. Hold the UNO near the hand you are exercising and flick (flex) a finger against the ring. Exaggerate the movement by bending and straightening the finger as much as you can. Do this 10 times slowly. Relax repeat 10 times more, or as instructed by a healthcare professional.

### **14. Finger Tip/Thumb Tip Flexion Pulls (USE EXERCISE LOOP)**

Hold UNO in the hand you are NOT exercising – with the exercise loop facing upward. Put the finger tip/thumb tip you want to exercise palm up, into the loop. Hold the UNO so there is a little tension on the ring. Bend (flex) the finger, pulling against the ring, holding for 3-5 seconds, then relax. Repeat 10 times slowly, or as instructed by a healthcare professional.

( CONTINUED )

### 15. Finger Adduction/Abduction (USE EXERCISE LOOP)

Hold UNO in the hand you are NOT exercising with the exercise loop to the side. Put the finger you want to exercise into the loop. Pull UNO so there is a little tension on the loop. While holding the finger straight, move the finger away from the UNO and hold 3-5 seconds. Repeat 10 times slowly, or as instructed by a healthcare professional. SWITCH direction of the UNO and repeat. Do this with each finger.

### 16. Fine Motor Dial Turns

Hold the UNO in the hand you are NOT exercising. Using the two fingers you want to exercise, turn the adjustment dial. While keeping the finger tips bent (flexed), slowly turn the dial 10 times in one direction – slowly – then 10 times the opposite way. Switch fingers and repeat.

### 17. Gross Grasp Presses – Table Top

Hold the UNO with a firm grasp with the flexion plunger facing down. Press the UNO down against the table top to compress the Flexion Plunger. Hold for 3-5 seconds, then relax. Repeat 10 times slowly, or as instructed by a healthcare professional.

### 18. Pinch Presses – Table Top (SWITCH FINGERS)

Hold the UNO with a firm pinch on the intended area of the adjustment dial with the flexion plunger facing down. Press the UNO down against the table top to compress the flexion plunger. Hold for 3-5 seconds, then relax. Repeat 10 times slowly, or as instructed by a healthcare professional, and then switch fingers.

### 19. Wrist Flexion/Extension – Gross Grasp Hold

Hold the UNO in your hand with a comfortable grip. With the forearm supported on the edge of a table, move the wrist up (extension) as far as you can and hold for 3-5 seconds. Then move the wrist down (flexion) as far as you can and hold for 3-5 seconds. Repeat 10 times slowly, or as instructed by a healthcare professional.

### 20. Forearm Rotation

Hold the UNO in your hand with a comfortable grip. Relax your arm by your side and bend (flex) the elbow about 90 degrees so the forearm is parallel to the floor. Turn your forearm so the palm faces up (supination) as much as you can and hold for 3-5 seconds. Then turn your forearm so the palm faces down (pronation) as much as you can and hold for 3-5 seconds. Repeat 10 times slowly, or as instructed by a healthcare professional.

### 21. Wrist – Lateral Motion – Ulnar (pinky side) Radial (thumb side) Deviation

Hold the UNO in your hand with a comfortable grip. Support your forearm with your other hand. Move your wrist to the pinky side, holding for 3 seconds, and then to the thumb side, holding for 3 seconds. Relax and repeat 10 times slowly, or as instructed by a healthcare professional.

( CONTINUED )

## 22. Fine Motor Twist Off

Hold the Center of the UNO in the fist of one hand and use the fingertips of the other hand to twist the loop-end to loosen it, and then the other way to tighten it. Repeat 10 times slowly, or as instructed by a healthcare professional.

## 23. Grasp Hold

Hold the center of the UNO in one hand and use the other hand to grasp and twist the loop end (using wrist motion) to loosen it and then the other direction to tighten it. Repeat 10 times slowly, or as instructed by a healthcare professional.

---

## HOW TO CHANGE YOUR VARIGRIP UNO SPRINGS:

VariGrip Uno features a screw cap, located towards the top of the unit that holds the spring in place. In order to change from one spring to the other, please follow the steps below:

Step 1  
Adjust resistance to the lowest setting.

Step 3  
Insert the other spring vertically inside the unit.

Step 2  
Turn the top cap counterclockwise until it pulls away from the VariGrip Unit. Please note that the spring that is already inside the unit may fall out into your hand.

Step 4  
Close and tighten the top cap by turning it clockwise until it stops moving.

---

## REPLACING THE EXERCISE LOOP:

VariGrip Uno also includes a screw cap, located towards the bottom of the unit that holds the elastic ring in place. In order to replace the elastic ring, please follow the steps below:

Step 1  
Adjust resistance to the lowest setting.

Step 4  
Insert the bottom of the replacement elastic in the center of the bottom twist-off cap.

Step 2  
Turn the bottom cap counterclockwise until it pulls away from the VariGrip Unit.

Step 5  
Pull the elastic by the "tail" and place the "ball" of the elastic ring inside the receptacle.

Step 3  
Remove the elastic ring "ball" from the receptacle.

Step 6  
Fold extra tail (past the "ball") so that it is facing the top of the elastic ring. Close and tighten the cap by turning it clockwise until it stops moving.

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D'Addario & Company, Inc. Farmingdale, NY 11735 USA. Designed in USA. Made in Canada. Dynatomy, Varigrip and D'Addario are registered trademarks or trademarks of D'Addario & Company, Inc. or its affiliates in the United States and/or other countries. Copyright © 2017. All rights reserved. Patent-Pending.
Dynatomy, Inc. • 595 Smith Street • Farmingdale, NY 11735